


Bipolar II[™] Hip System




Bipolar II Features

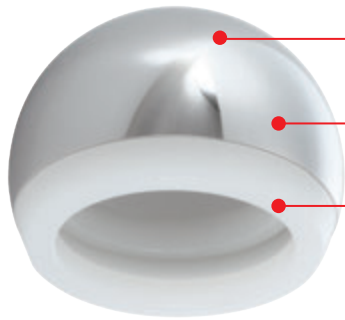

Cap sizes available from 38 to 56 mm in 1 mm size increment


Highly polish surface to reduce the wear of acetabulum*

One - piece assembly

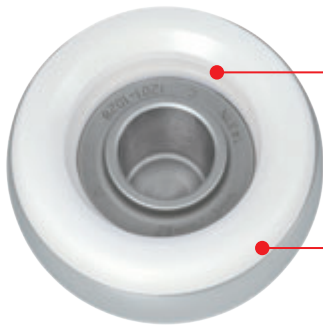

Simple locking ring mechanism enhances surgical efficiency, while maintaining strength

Minimum PE thickness of 5 mm

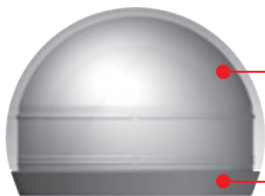

Anti-rotation design minimizes micromotion between the cap and the PE liner

Outer beveled lip design allows for easy reduction

Devoted *to*
Orthopedics

*Metal surface roughness of $Ra \leq 0.02 \mu\text{m}$ to fulfill the ISO Standard of $Ra \leq 0.5 \mu\text{m}$

Surgical Techniques

Sizing the Acetabulum




Trial Assembly and Reduction




Cap trial sizes	38, 39, 40 mm	41, 42 mm	43~56 mm
Head trial sizes	22 mm	22/26 mm	28 mm

Trial Disassembly

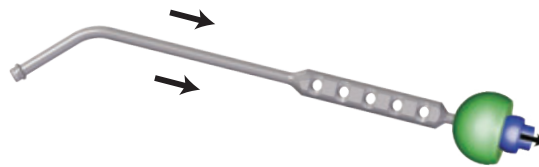

Implant Assembly

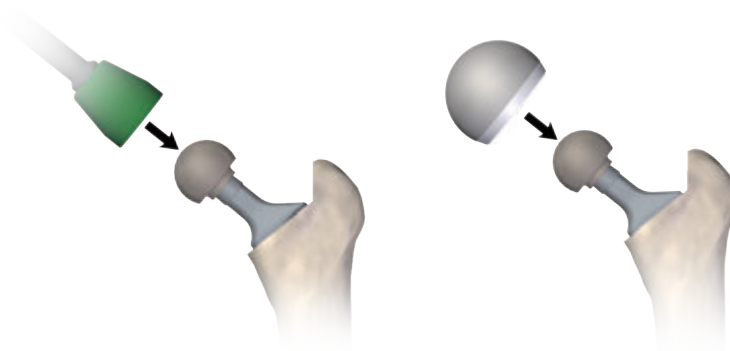

Implant Disassembly



Bipolar II Instruments


★ *Special Order Items*

Femoral Head Trial



Cat. No.	Description
1201 - 2122 - RB	Ø 22mm +0 mm
1201 - 2322 - RB	Ø 22mm +3 mm
1201 - 2522 - RB	Ø 22mm +6 mm
1201 - 2722 - RB	Ø 22mm +9 mm
1201 - 2126 - RB	Ø 26mm +0 mm
1201 - 2326 - RB	Ø 26mm +3 mm
1201 - 2526 - RB	Ø 26mm +6 mm
1201 - 2726 - RB	Ø 26mm +9 mm
1201 - 2028 - RB	Ø 28mm -3 mm
1201 - 2128 - RB	Ø 28mm +0 mm
1201 - 2428 - RB	Ø 28mm +5 mm
1201 - 2828 - RB	Ø 28mm +10 mm

U2 Bipolar Trial

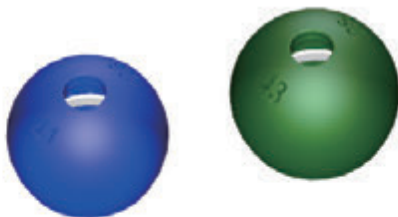

Cat. No.	Description
1503-4338	Ø 38 mm
1503-4339	Ø 39 mm
1503-4340	Ø 40 mm
★ 1503-4341	Ø 41 mm
★ 1503-4342	Ø 42 mm
1503-4041	Ø 41 mm
1503-4042	Ø 42 mm
1503-4143	Ø 43 mm
1503-4144	Ø 44 mm
1503-4145	Ø 45 mm
1503-4146	Ø 46 mm
1503-4147	Ø 47 mm
1503-4148	Ø 48 mm
1503-4149	Ø 49 mm
1503-4150	Ø 50 mm
1503-4151	Ø 51 mm
1503-4152	Ø 52 mm
1503-4153	Ø 53 mm
1503-4154	Ø 54 mm
1503-4155	Ø 55 mm
1503-4156	Ø 56 mm

ID 22mm (applies to Ø 38 mm to Ø 42 mm)
 ID 26 mm (applies to Ø 41 mm to Ø 42 mm)
 ID 28 mm (applies to Ø 43 mm to Ø 56 mm)

Head Remover



Cat. No.	Description
9203 - 5022	22 mm
9203 - 5026	26 mm
9203 - 5028	28 mm

Bipolar II Instruments




Cat. No.	Description
9206 - 1110	Universal Handle, 140 mm
9206 - 1112	Universal Handle, 230 mm



Cat. No.	Description
9204 - 1004	Bipolar Trial Handle



Cat. No.	Description
9204 - 1222 - RA	Femoral Head Impactor, 22mm

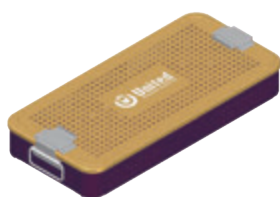

Cat. No.	Description
9204 - 8041 - RC	Bipolar II Case

Bipolar II Implant

★ Special Order Items



Bipolar Cap



Metal Head

Cat. No.	Outer Diameter	Cat. No.	Description	Head Size
1503 - 3338	38 mm	1201 - 1122	+0 mm	22 mm
1503 - 3339	39 mm	1201 - 1322	+3 mm	
1503 - 3340	40 mm	1201 - 1522	+6 mm	
★1503 - 3341	41 mm	1201 - 1722	+9 mm	
★1503 - 3342	42 mm			
1503 - 3041	41 mm	1201 - 1126	+0 mm	26 mm
1503 - 3042	42 mm	1201 - 1326	+3 mm	
		1201 - 1526	+6 mm	
		1201 - 1726	+9 mm	
1503 - 3143	43 mm	1201 - 1028	-3 mm	28 mm
1503 - 3144	44 mm			
1503 - 3145	45 mm	1201 - 1128	+0 mm	
1503 - 3146	46 mm			
1503 - 3147	47 mm	1201 - 1428	+5 mm	
1503 - 3148	48 mm			
1503 - 3149	49 mm	1201 - 1828	+10 mm	
1503 - 3150	50 mm			
1503 - 3151	51 mm			
1503 - 3152	52 mm			
1503 - 3153	53 mm			
1503 - 3154	54 mm			
1503 - 3155	55 mm			
1503 - 3156	56 mm			



Office

■ Taiwan

United Orthopedic Corporation

12F, No. 80, Sec. 1, Chenggong Rd, Yonghe Dist.,

New Taipei City 23452, Taiwan

Tel: +886 2 2929-4567 Fax: +886 2 2922-4567

■ USA

UOC USA Inc.

20 Fairbanks, Suite 173, Irvine, CA 92618

Tel: +1 949-328-3366 Fax: +1 949-328-3367

■ China

United Medical Instrument Co. Ltd.

Room 3-1-201, No.45, Jiangchang First Road, Shanghai, China

Tel: +86 21 3661-8055 Fax: +86 21 6628-3031

■ Europe

UOC Europe Holding SA

Avenue Général Guisan 60A

1009 Pully, Switzerland

Tel: +41 79 774 89 33



www.uoc.com.tw