



U2 KNEE MIS

Posterior Reference System

This Surgical Instrument is developed in conjunction with

Ing-Ho, Chen M.D.

*President of Buddhist Tzu-Chi General Hospital
Taipei, Taiwan*

The U2 Knee MIS technique cooperates with several special designed user-friendly instruments to achieve intraoperative SAFTY and COMFORT and excellent postoperative result with minimal learning curve. To enhance field of vision and to reduce the tension of surrounding soft tissue, it is essential to change different positions of the lower limb during each steps.




Femoral Intramedullary Alignment Guide

-Adjustable Angle 0-11°



-Fixed Angle 5°, 7°, 9°



That can help to accomplish distal femoral cutting guide set under nearly extension to reduce anterior tension.

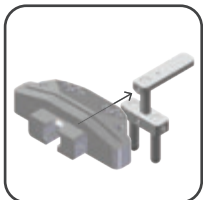

Distal Femoral Alignment Guide & Cutting Guide

■ Adjustable Resection Level

Slide the cutting guide into anterior portion of wound via the track attached on femoral alignment guide; and cutting level can be read on top of the track.

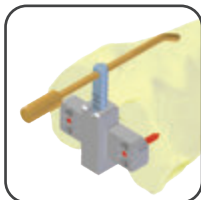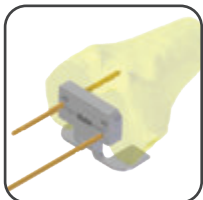

External Rotation Guide and Sizer

■ Two Pieces Design, Measure the AP Size of Femur with ease

The two pieces sizer can be easily placed onto distal cutting surface to obtain accurate AP dimension of femur in limited field of vision.



Femoral A/P Chamfer Cutting Guide

Built-in fixation pegs; with the ML width comparable with actual implant.















Open-Slot Tibial Cutting Guide

Allow proximal tibial cutting through medial to lateral, enhanced work efficiency.



U2 KNEE MIS Instruments Set

Picture	Part No.	Description
	9303-2105	Femoral IM Alignment Guide
	9303-2345	Femoral IM Alignment Guide , 5°, Right
	9303-2347	Femoral IM Alignment Guide , 7°, Right
	9303-2349	Femoral IM Alignment Guide , 9°, Right
	9303-2335	Femoral IM Alignment Guide , 5°, Left
	9303-2337	Femoral IM Alignment Guide , 7°, Left
	9303-2339	Femoral IM Alignment Guide , 9°, Left
	9303-2106	Distal Femoral Alignment Guide
	9303-2107	Distal Femoral Cutting Guide
	9303-2409-RD	External Rotation Guide, 3°x 9mm
	9303-7302-RE	Femoral Sizer, Posterior Ref.
	9303-2581	Femoral A/P Chamfer Cutting Guide, #1
	9303-2582	Femoral A/P Chamfer Cutting Guide, #2
	9303-2583	Femoral A/P Chamfer Cutting Guide, #3
	9303-2584	Femoral A/P Chamfer Cutting Guide, #4
	9303-2585	Femoral A/P Chamfer Cutting Guide, #5
	9303-2586	Femoral A/P Chamfer Cutting Guide, #6 (Option)
	9303-5023	External rotation guide driver
	9403-2108	Tibial EM Alignment Guide
	9403-2140	Tibial Cutting Jig, Open Slot, 0°, Left
	9403-2240	Tibial Cutting Jig, Open Slot, 0°, Right
	9303-8060-RA	U2 KNEE MIS Case
	9303-5301	Chamfer Guide Extractor